


# CPAP CLEANER

## Intended use



This cpap cleaner makes sterilization of the respirator, pipe, mask and humidifier.

## Main components of the host

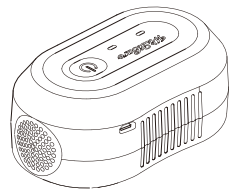

Type:WQ2200

Operating instruction

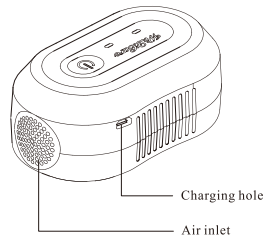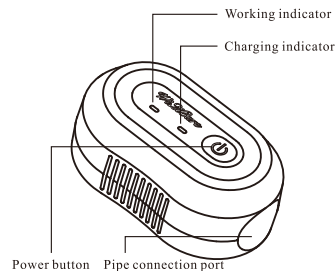

## Product formation

Host, charging line, instruction manual


Note: This product has a built-in rechargeable lithium battery, which requires the output current of the charging line and charger. The CPAP CLEANER can be charged by the mobile phone adapter, Or charge it with the same power charging head.

## Technical parameters

Size	105x60x42mm
Weight	195G
Built-in rechargeable lithium battery	2200mAh
Working voltage	Dc3.6V-4.3V
Storage environment	-5°C- 45°C
Use environment	-20°C- 55°C
Classification of waterproof	IPX1

Disinfection machine works for about 25 minutes to disinfect the product effectively and turn off the machine automatically, without the need to lengthen the disinfection time.

## Disinfection diagram of the disinfection machine:

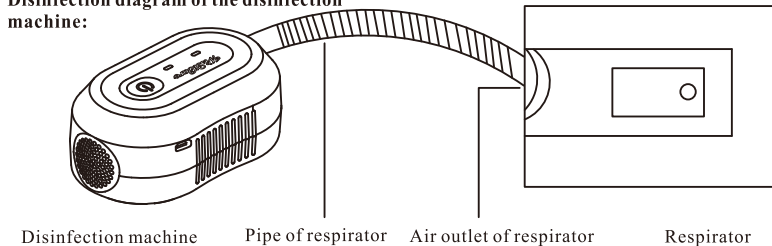

## Operation description

Before using, please unplug the power of the respirator, clear the water of the humidifier and connect the pipe. In order to improve the quality of disinfection, it is best to be placed in a closed space for disinfection, and you can choose the portable bag of the respirator.

**Turn on:** press the switch button longly, the buzzer rings 1 time (Short sound), and the work indicator lights (green), then the disinfection machine begins to work.

**Operation:** after the disinfection machine begins to work, it begins to produce ozone. In the course of operation, the work indicator keeps lighting and no other human operation is needed.



E-mail: cpapcleaner@163.com

Tel Tel +086-022-58655869

**Turn off:** After the end of the disinfectant work, the buzzer rings 3 times(interval), the machine automatically turns off , and the work light extinguishes.

If the work process needs to be terminated, the power button can be pressed longly and the machine will be turned off manually. The buzzer will ring 1 time before the shutdown (long sound), suggesting that it has been turned off, the work lights go out work.

### Charging

This product has a built-in rechargeable lithium battery, and the charging connection diagram is as follows.

Charging: in order to ensure the best charging effect, please charge in the state of off. The charging indicator is red during the charging process ,and the indicator is become green after charging is completed (light green).It takes about 10 hours to complete the charge. The performance of the rechargeable battery will decline with the increasing times of using , which is a normal phenomenon. Low power reminding: in the course of work, if the battery power is too low, the work indicator will turn green and keep flashing. At this time, if the charge is not carried out, the disinfection machine can still work

normally, but the residual quantity can not guarantee the completion of a disinfection process.

### Matters needing attention:

- When the humidifier is disinfected, be sure to dry the water in the tank before disinfecting.
- After disinfection, please use the ventilator 1 hours later so that the ozone can dissipate completely.
- If there is still a smell of ozone when the ventilator is used after disinfection. You can open the respirator and empty the residual ozone odor.
- The concentration of ozone is produced by the host of the disinfectant. It is determined by the volume of the bag and the time of disinfection machine. This product is strictly sterilizing and formulate the standard of disinfection. Do not try to increase the time and the concentration of disinfection, excessive concentration will cause oxidation of the machine, please do not use any non - standard method of detoxification.
- The built-in lithium battery capacity of the machine is in accordance with the requirements of the civil aircraft and can be carried on with you.
- Please do not disassemble the machine by yourself.

Please contact the service provider or agent after the failure of the machine. The damage caused by dismantling is not within the scope of warranty.

●In order to ensure the health of the patient, it is suggested that should disinfect the respirator and mask one month.

### Quality assurance:

The product can be replaced within 1 months, the host can be guaranteed for 1 years, and 90 days for the accessories for the damage caused by nonhuman factors(The date of warranty is the date that the product is sole to the user by the dealer)


### Failure recovery

Failure	Recovery
The machine can't start when press the power button	Connect charger to charge
Charging indicator can not light	Please check that whether the adapter is connected normally
The machine stops working without alarm	Connect charger to charge
Can not turn on the machine by pressing the power button after charging	Please check whether the charging connection is normal, if you still can not turn on it, send it for repairing
After the machine is switched on, it can not stop automatically	Reopen the machine again and again, if this happens repeatedly, then send it for repairing
No ozone generated	1. There is no reaction when press the button, and the fan does not turn. Check if it needs to be charged. 2. Check whether the switch is in the working state and reopen the machine. 3. The fan turns and the indicator lights, then send it for repairing
When using The green light flashes, and issues an alarm of "didi"	The power is too low and connect the charger to charge.

How to sterilize the mask?

1. Place the mask and sterilizer in a sealed plastic bag or in a closed backpack.


2. Then turn on the disinfection machine, the disinfection machine will work automatically for 25 minutes, and the disinfection process will be completed.

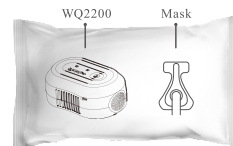

How to sterilize the hose and CPAP equipment?

1, connect the hose with CPAP and sterilizer as shown below

2. Turn on the sterilizer and run it automatically for 25 minutes. The disinfection process is completed. We recommend continuing to sterilize the inside of the CPAP machine within 30 days.

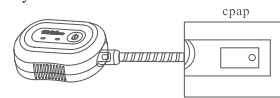

### Certificate

Product name: Respirator Disinfection Machine

Product type: WQ2200

Checker: QC00

Date of checking: 2018-10-15

E-mail : cpapcleaner@163.com

### Kindly reminder

This instruction provides guidance on the use of disinfectant. Please read the instructions in detail before using the product to ensure the safe use of the product. This instruction is attached with a warranty card and a certificate of qualification as maintenance record, please keep it in good care.